

# TROUBLESHOOTING



TOPCON®

## CT-80 (also applies to CT-60)

The Topcon CT-80 Computerized (Non-Contact) Tonometer brings quick and accurate tonometry readings to your fingertips.



### 1. Monitor screen does not work.



- Secure power cable in AC plug.



- Plug power cable into machine body.



- Check the fuse.

### 2. Paper does not come out.

- Check paper supply. Ensure only Topcon paper is used (Part #42330 9003B).
- Check for paper jam and remove if jammed.

### 3. Machine body does not move.

- Check Clamping Knob under instrument. Loosen if found fastened.
- Check Fixing Knob near the joystick. Turn to the left to loosen; turn to the right to fasten.





#### 4. Unit has high readings and error messages.

- Clean inside the measuring nozzle. To clean, obtain tiny Topcon q-tips (Part #41601 86060).

**Warning: regular size q-tips can break the nozzle.**

Denatured alcohol must be used. Note: this can be purchased at any local hardware store.

**Warning: Regular alcohol must not be used.**



Moisten the Topcon q-tip with solution. Insert the q-tip into the nozzle. Lightly touch the glass surface while turning the q-tip. Replace the q-tip with a new one and wipe the surface in a similar fashion. Cleaning is completed when the grease is thoroughly removed and the fixation target is clearly seen.

- Clean measuring window.

